

Instructions for use - Please read carefully!

Blanchipon®

Concentrate

Universal concentrated detergent for medical devices
(for washable surfaces and objects)

For professional use

Composition

Anionic surfactants (15 – 30%)

Range of application

- Detergent for washable **surfaces** of all types for ex. textile-free floors, stairs, corridors, walls, tiles, doors, window frames, radiators, **equipment** and **apparatus**, sanitary equipment, fridges etc.
- Also for use in kitchen areas

Mode of application / dosage

Blanchipon® is a liquid concentrate with which any use-dilution required can be prepared by adding water (at max. room temperature). Always fill up container with water first, and then add **Blanchipon®**.

Cleaning of surfaces

Normal and dried dirt 0.5 %
Heavy soil 1.0 %

Concentration	0.5 %	1.0%
Dosing system (for 8 l of water)		
Dosing pump (1 stroke = 20 ml)	2 strokes	4 strokes
Dosing cap	2 caps	4 caps

In case of use of other amounts for the use-dilution and other concentrations: see dosing table.

Warnings / storage

Use diluted only!

Keep **Blanchipon** in a cool but frost free and dry place and out of reach of children. Wear protective gloves and suitable tight-fitting goggles when handling the product.

Properties

- Highly concentrated detergent for the gentle cleaning of surfaces, equipment and instruments; well tolerated by the skin.
- Eliminates big amounts of dirt of organic origin especially blood and serum
- Fresh, non clinging scent
- Biodegradable
- Resists hard water
- Mixable with the following surface disinfectants: , **Aldasan® 2000**, **Lysoformin®** and **Trichlorol®**.

Special precautions

Use this medical device safely. Always read the label and product information before use!

Standard packaging / transport units:

12 x 1000 ml bottles
2 x 5 l can

article number

6076
5025

Accessories: dosing pump for 5 l can (1 stroke = 20 ml)

The updated Safety Data Sheet and the Operating Instruction Sheet are to be found under www.lysoform.de.

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D – 12247 Berlin
Tel.: +49-30-77992-0 Telefax: +49-30-77992-219

1216-9