



SURFACE
DISINFECTION

Fugaten®-Spray

Ready-to-use spray disinfection based on alcohol
for rapid disinfection of surfaces, objects and medical devices



CE0482

Product Information

Please read carefully!

Fugaten®-Spray

Ready for use
For professional use.

BAuA: Reg-Nr. N-12739, N-12741

INGREDIENTS

100 g Fugaten®-Spray contains:
70 g Ethanol (denatured)

SCOPE OF APPLICATION

For rapid disinfection of

- Small areas, objects and medical devices (e.g., operating tables, examination chairs).
- Hard-to-reach areas.
- Other alcohol-resistant materials.
- Unperfumed version especially well-suited to the food industry.

INSTRUCTIONS FOR USE/DOSAGE

Use undiluted.

Spray Fugaten®-Spray from a distance of approx. 30 cm.

The surfaces and objects must be moistened completely (max. 50 ml per m² room floor area).

Allow to dry before contact with the disinfected surfaces.

surface disinfection acc. to VAH-methods 2015, clean conditions

bactericidal, yeasticidal	100 % – 5 min
---------------------------	---------------

European Standards

EN 13697 bactericidal, yeasticidal (clean conditions)	100 % – 5 min
---	---------------

EN 16615 bactericidal, yeasticidal (clean conditions)	100 % – 1 min
---	---------------

EN 13624 fungicidal (clean and dirty conditions)	100 % – 1 min
--	---------------

EN 14348 mycobactericidal (dirty conditions)	100 % – 30 s
--	--------------

EN 14476 virucidal against enveloped viruses* (incl. HIV, HBV, HCV) (dirty conditions)	100 % – 30 s
--	--------------

EN 14476 limited spectrum virucidal activity* (clean conditions)	100 % – 1 min
--	---------------

EN 14476 virucidal* (clean conditions)	100 % – 5 min
--	---------------

*acc. to recommendations of Robert-Koch-Institute Berlin, published in Federal Health Gazette 3/2017

WARNING AND STORAGE NOTICES

- Fugaten®-Spray is highly flammable!
Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke.
Application max. 50 ml/m².
- Keep disinfectant at a cool, but frost-free, well ventilated and dry place, away from sources of ignition and out of the reach of children.
- Do not use after expiration date.

PRODUCT FEATURES

- Spectrum of efficacy: Bactericidal (incl. TBC), fungicidal, virucidal.
- Rapid onset of action, dries quickly after disinfection of surfaces and objects.
- Fresh scent, deodorizes room air.
- Good wetting ability.
- Compatible with most materials.
- Ready-for-use product in refillable PE bottle with spray head.

SPECIAL NOTICES

a) Countermeasures in case of improper use

Fugaten®-Spray acts as an irritant on contact with mucous membranes (eyes, throat, stomach, intestines, etc.). Dilution of the preparation is recommended as an immediate measure in case of such irritations. Drink plenty of water when ingested orally, rinse with plenty of water in case of contact with other mucous membranes. In either scenario, a doctor should be consulted immediately.

b) Additional notices

Do not spray on materials sensitive to alcohol (paints, stains, plexiglas, etc.).

c) Use Fugaten®-Spray safely. Always read the label and product information before use.

Certified and registered in all relevant listings.

SHIPPING UNIT/CONTAINER

	Art. no.
10 x 500 ml bottle with spray head	2058
12 x 1000 ml bottle with spray head	2292
12 x 1000 ml with spray head (unperfumed)	2272
2 x 5 litre canister	9825
2 x 5 litre canister (unperfumed)	9823

LYSOFORM
Seit ▪ Since ▪ Depuis 1900

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D-12247 Berlin
Telefon +49 30 77992-0
www.lysoform.com, e-mail: kontakt@lysoform.de