

Instructions for use - Please read carefully!

Aerodesin®

Ready for use

Disinfectant ready-for-use based on alcohol for a rapid disinfection of medical devices.

For professional use

Composition

100 g solution contains:
32.5 g Propan-1-ol, 18.0 g Ethanol (denatured)

Range of application

Rapid disinfection of

- Small surfaces and objects

Mode of application / Dosage

Use undiluted only!

Apply and rub **Aerodesin®** on objects and surfaces with a dry cloth. Make sure that the product has been evenly applied on the objects and surfaces and that these are evenly wetted (max. 50 ml/ m² of the room surface)

Disinfection by wiping according to the VAH procedure of 2015, with mechanical effect, dirty conditions		
bactericidal, levuricidal, tuberculicidal	100%	60 s
Tests according to the directive of DVV/RKI		
virucidal against enveloped viruses (incl. HIV, HBV, HCV) according to the recommendations of RKI of 2017	100%	30 s
Euro Norms (dirty conditions)		
EN 13624, fungicidal	100%	5 mn

Warnings / Storage

- **Aerodesin®** is inflammable!
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke.
- Keep disinfectants in a cool but frost free, well ventilated and dry place, away from ignition sources and out of reach of children.
- Do not use beyond expiry date.

Properties

- Control of bacteria (incl. tb), fungi and enveloped viruses
- Rapid action and drying after applying on surfaces and objects
- Freshly scented
- Good wetting power

Special precautions

a) Counter-measures in case of improper use:

Aerodesin® is irritating in contact with the mucous membranes (eyes, throat, stomach, bowels etc.). In case of such irritations, dilute product immediately. If it has been **swallowed**, make patient drink **plenty** of water; after contact with other mucous membranes, rinse thoroughly with plenty of water. In all cases get medical attention immediately.

b) Miscellaneous:

Do not apply on materials that are not compatible with alcohol (coated, pickled surfaces, Plexiglas etc.).

c) Use disinfectant safely. Always read the label and product information before use.

Certified and mentioned in the relevant lists.

Standard packaging / transport units:

12 x 1 l bottle

article No: 9896

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D – 12247 Berlin
Tel.: +49-30-77992-0 Telefax: +49-30-77992-219
www.lysoform.de, e-mail: kontakt@lysoform.de

0119-9