



SURFACE
DISINFECTANTS



HAND
CLEANING

Hospisept®-Wipes

Disinfection and cleaning wipes for surfaces,
inventory and medical devices
and for hand cleaning



CE0482

Product Information

Please read carefully!

Hospisept®-Wipes

BAuA: Reg.-Nr.: N-12666, N-12667, N-12670

INGREDIENTS

1 impregnated wipe contains on average:
1,1 g Propan-1-ol, 0.32 g Ethanol 96 %, Butan-2-on,
Dibutyladipate, purified water

SCOPE OF APPLICATION

- For the disinfection and cleaning of small surfaces and the disinfection of medical devices, such as contact lenses for diagnostic purposes, dental units etc., and for antimicrobial precleaning of endoscopes.
- Applicable everywhere where spraying is not possible and rapid drying is important.
- hand cleaning

INSTRUCTIONS FOR USE/DOSAGE

- Wipe the surface (take care to ensure complete moistening of the surface) and let the product operate until dry (ca. 5 minutes).
 - limited spectrum virucidal activity* incl. HBV/HCV/HIV and Rota and Norovirus (MNV): 30 s
Adenovirus: 30 s
Polyomavirus (SV 40): 30 s
- * acc. to recommendations of the Robert-Koch-Institute (RKI) Germany, Federal Health Gazette 3/2017.

WARNINGS AND STORAGE NOTICES

- Flammable! (Flash point according to DIN 51755: 26 °C).
- Store the medical device in a cool but frost-free and dry place, away from sources of ignition and out of the reach of children.
- Avoid contact with the eyes.
- Do not treat Alcohol-sensitive plastics such as acrylic glass.
- Shelf-life after first opening of the product: 6 months
- Do not use after expiration date.

PRODUCT FEATURES

- Broad spectrum of efficacy: Efficacious against bacteria (including TBC), yeasts, enveloped viruses and rotaviruses
- Wipes tear off individually.
- Can be used immediately.
- Well tolerated by the skin, surfactant-free and fragrance-free.
- In contrast to other surface cleaning and disinfection wipes, with whom according to BGR 250 / TRBA 250 gloves must be worn, Hospisept® Wipes can be used with unprotected hands.
- Suitable for any first aid kit (ambulance, rescue helicopter).
- No hazardous waste after use.

SPECIAL NOTICES

Use medical devices safely! Always read the label and product information before use.

SHIPPING UNIT/CONTAINER

	Art no.
6 canisters with 100 wipes each	3585
6 refill bags with 100 wipes each	3575
15 individually packaged Hospisept wipes in cardboard container	3595

LYSOFORM
Seit ▪ Since ▪ Depuis 1900

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D-12247 Berlin
Telefon +49 30 77992-0
www.lysoform.com, e-mail: kontakt@lysoform.de