



SURFACE
DISINFECTION

Lysoformin® Plus- Foam

Fast-acting foam for disinfection
and cleaning of medical inventory.



CE0482

Product Information

Please read carefully!

Lysoformin® Plus-Foam

Ready for use.
For professional use.

BAuA: Reg-Nr. N-76300

INGREDIENTS

100 g Lysoformin® Plus-Foam contains:
0.18 g Polyhexamethylene biguanide · HCl,
0.46 g Bis-(3-aminopropyl) dodecylamine,
0.10 g Didecylidimethylammonium chloride, perfume

SCOPE OF APPLICATION

Fast-acting disinfectant for medical inventory acc. to German Medical Devices Act (MPDG) and Medical Device Regulation (MDR). Also suitable for use on other types of water-resistant surfaces.

Suitable for the **disinfection** and **cleaning** in separate steps of surfaces (also of medical devices). Only use **Lysoformin® Plus-Foam** in the manner indicated in the instructions for use provided by the manufacturer.

INSTRUCTIONS FOR USE/DOSAGE

Ready for use. Cover the surfaces intended for disinfection with foam, and allow the product to operate. Afterwards, wipe the surface with a clean, moist cloth, or spray **Lysoformin® Plus-Foam** on a disposable cloth and wipe the surface.

Application time: 5 minutes.

Virucidal against enveloped viruses, incl. HBV/HCV/HIV* 5 min.
Polyomavirus (SV 40) 15 min.

* acc. to recommendations of the Robert-Koch-Institute (RKI)
Germany, Federal Health Gazette 3/2017.

WARNING AND STORAGE NOTICES

- Do not breathe in spray mist, wear respiratory protection if necessary.
- Wear protective gloves and protective clothing.
- Do not use after expiration date.
- Keep disinfectant at a cool but frost-free, well ventilated and dry place and out of reach of children.

PRODUCT FEATURES

- Efficacious against bacteria, yeast, and enveloped viruses.
- Optimal moistening of surfaces, resulting in efficient and safe disinfection.
- Easy application, ready for use.
- Fresh scent.

SPECIAL NOTICES

a) Countermeasures in case of improper use

Should, through improper use, the product Lysoformin® Plus-Foam have come into contact with any mucous membranes (e.g., eyes, throat, or stomach), a doctor should be consulted immediately.

As an immediate measure, dilution of the preparation is recommended. In case of oral ingestion, drink plenty of water. In case of contact with the eyes, rinse thoroughly with generous amounts of water.

b) Exercise caution when handling Lysoformin® Plus-Foam!

Always read the label and product information before handling the product.

c) Additional notice

Surfaces and objects that come into contact with foodstuffs should be rinsed off with drinking water following disinfection.

SHIPPING UNIT/CONTAINER

12 x 1 litre bottle with spray head
2 x 5 litre canister

Art. no.

2505
2509

LYSOFORM
Seit ▪ Since ▪ Depuis 1900

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D-12247 Berlin
Telefon +49 30 77992-0
www.lysoform.com, e-mail: kontakt@lysoform.de

2022/10/31_1